

**10A NCAC 41B .0322 INTOXIMETERS: MODEL INTOX EC/IR II AND MODEL INTOX EC/IR II
(ENHANCED WITH SERIAL NUMBER 10,000 OR HIGHER)**

The following operational procedures shall be followed when administering a test using the Intoximeters, Model Intox EC/IR II and Model Intox EC/IR II (Enhanced with serial number 10,000 or higher):

- (1) Ensure instrument displays time and date;
- (2) Ensure observation period requirements have been met;
- (3) Initiate breath test sequence;
- (4) Enter information as prompted by the instrument;
- (5) Verify instrument accuracy;
- (6) When "PLEASE BLOW" appears, collect breath sample;
- (7) When "PLEASE BLOW" appears, collect breath sample; and
- (8) Print test record.

If the alcohol concentrations differ by more than 0.02, a third or fourth breath sample shall be collected when "PLEASE BLOW" appears. Sequential tests shall be administered as soon as feasible by repeating steps (1) through (8), as applicable.

*History Note: Authority G.S. 15A-534.2(d); 20-139.1(b);
Eff. November 1, 2007;
Pursuant to G.S. 150B-21.3A, rule is necessary without substantive public interest Eff. January 9,
2018;
Amended Eff. April 1, 2020.*